

VoiceQast Kit SN: VCK01


Our all included audio kit, designed for training scenarios!

Broadcast your transformed voice to a portable speaker or an agent earpiece within seconds.

The tablet comes fully-prefconfigured with the voice changer and the Innov2Learn® mobile APP including the VoiceQast® feature.

Assemble and use it straight out of the box!

WHAT'S INCLUDED IN THE CASE

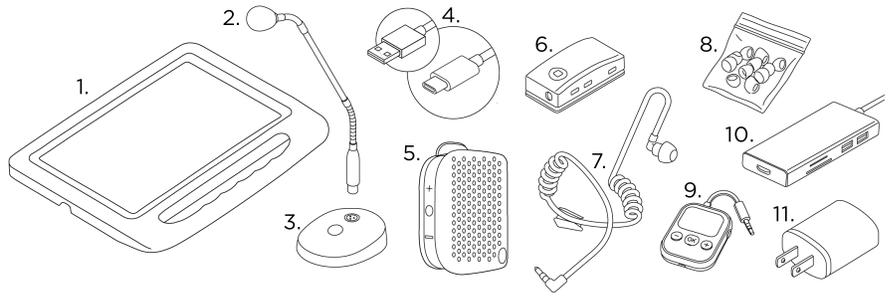

- | | |
|--|--------------------------|
| 1. Preconfigured Tablet | 6. Bluetooth receiver |
| 2. Microphone | 7. Earpiece |
| 3. Microphone Base | 8. Earpiece tips (x50) |
| 4. USB Type-A to USB-C Cable (x3) + USB-C to USB-C Cable (x1) | 9. Bluetooth Transmitter |
| 5. Monster Speaker | 10. USB Hub |
| | 11. USB Chargers |

ENHANCED COMMUNICATION




Say goodbye to simulation limitations!



Your standardized patients can now receive additional instructions clearly and securely through the earpiece.



No need to strain your voice anymore—the voice changer does it for you! Choose between a child's voice, a masculine tone, or a feminine one—it's up to you!

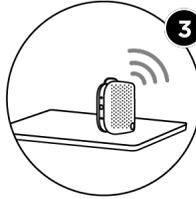


Broadcast color-coded messages in any room!

VOICEQAST


VoiceQast is built right into the free Innov2Learn app.

Download the app from the App Store or Google Play, or visit our website:


innov2learn.ca/app/

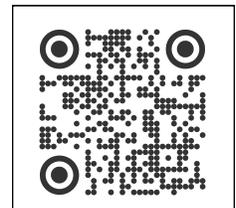

BATTERY-POWERED SETUP

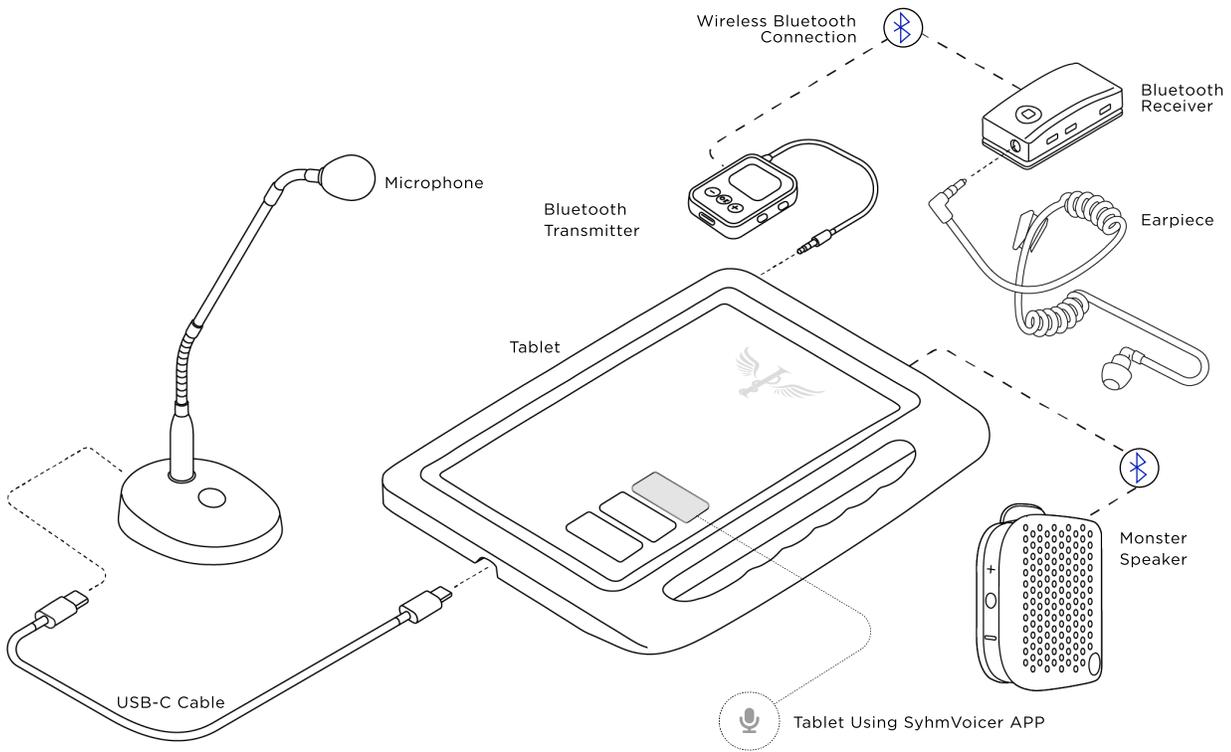


WALL POWERED SETUP

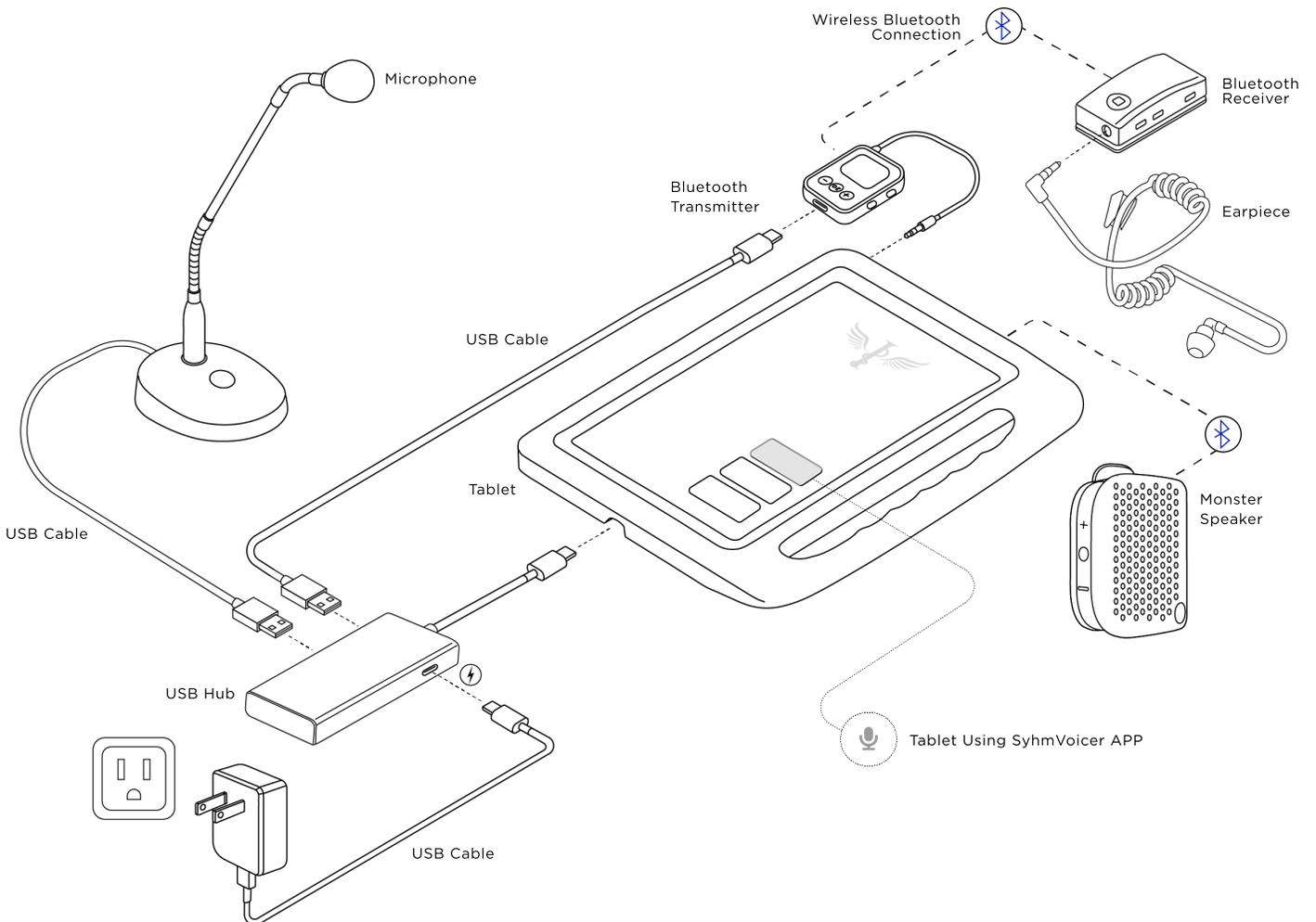

BLUETOOTH RECEIVER


Getting Started

1. Power on: Press and hold for three seconds until the button lights up blue.
2. Pairing: Press and hold for three seconds until the light flashes, then select "BTR3" from the available Bluetooth devices on the tablet.
3. Plug the earpiece into the receiver.

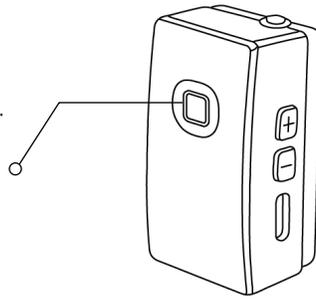

- Volume can be adjusted using the red (+ / -) buttons on the receiver.
- Secure one-way communication from instructor to participant.

- Charge using a USB-C cable.

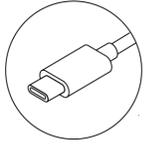


MONSTER SPEAKER

Your manikins' "new" voice and/or your portable system for broadcasting announcements.


Getting Started

1. Press and hold for two seconds to power on/off.
2. Pairing: Hold the button until the blue light flashes, then select the speaker from the list of visible Bluetooth devices on your phone or tablet.

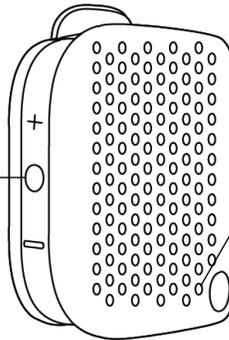

- Red light: Charging
- Light off: Charging complete
- Flashing blue light: Pairing mode
- Blue light: Connected


- Charge using a USB-C cable.

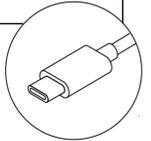

SYHMVOICER APP INTERFACE



SyhmVoicer

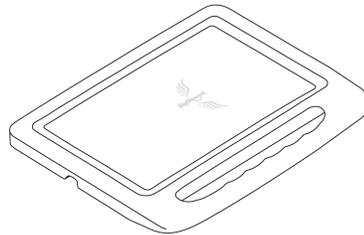

Getting Started

1. Power ON the receiver and/or speaker using Android Bluetooth icon
2. Start the voice changer by double-clicking the SyhmVoicer Icon
3. Select desired voice by clicking the corresponding character.

