

- ① **Download**
 - The Innov2learn app from the Play Store or App Store.

- ② **Turn on the device**
 - Press and hold the power button until something appears on the screen.*
 - For the OXI, a single press is enough.
 - * *Make sure to unplug the device before using it.*

- ③ **Enable Bluetooth**
 - Grant all permissions requested by the app.
 - Open the app & click on “Connect.”

- ④ **Select**
 - The devices you want to pair.
 - Tap the icon at the bottom right to secure the connection.

- ⑤ **Start your simulation**
 - Program a reading (Preset Mode) for the Gluco range, Oxi, Forehead-T, and Thermo range (if equipped with the Thermo SP Trigger) when no detection has occurred. Finalize the value entry by clicking “Preset.”

 - For real-time readings (when detection occurs, or for CO2-Detect, SpotV II, and the Thermo range WITHOUT the Thermo SP Trigger), send the entered values directly by selecting the send icon.


① Download Innov2Learn App




② Turn on the device



③ Enable Bluetooth

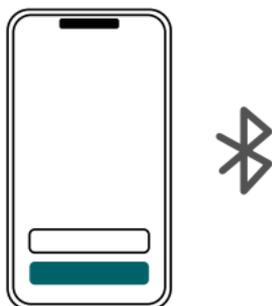

④ Select

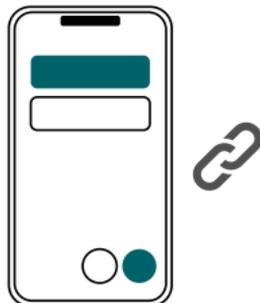

⑤ Start your simulation

