



SP Thermo Trigger - Instructions

1

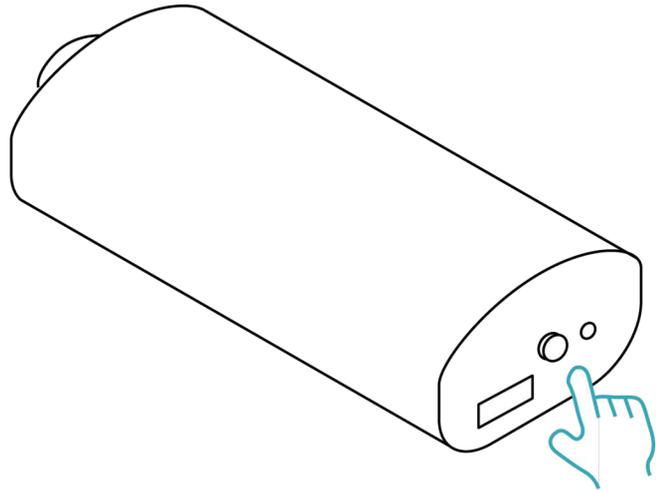


Press the small power button next to the charging port to turn on the SP Thermo Trigger.

2

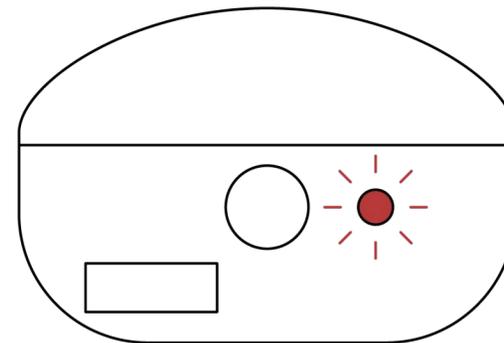


The red LED lights up when the device is powered on.

3

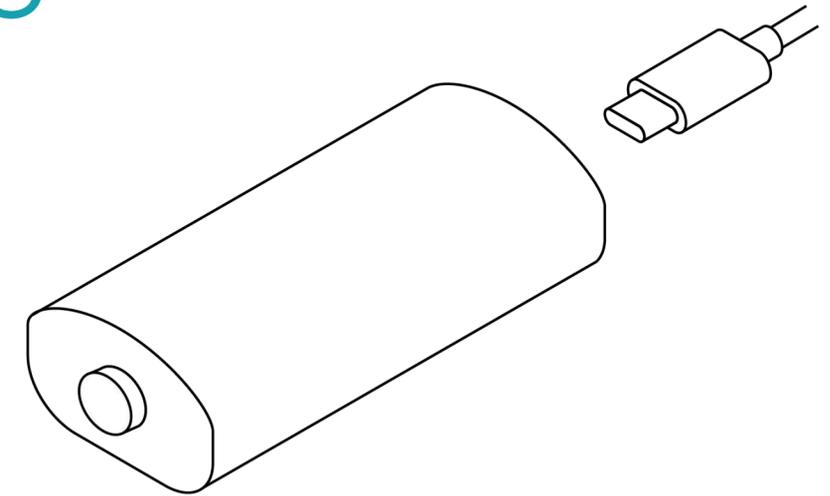

Charge the SP Thermo Trigger using the USB-C port.

4

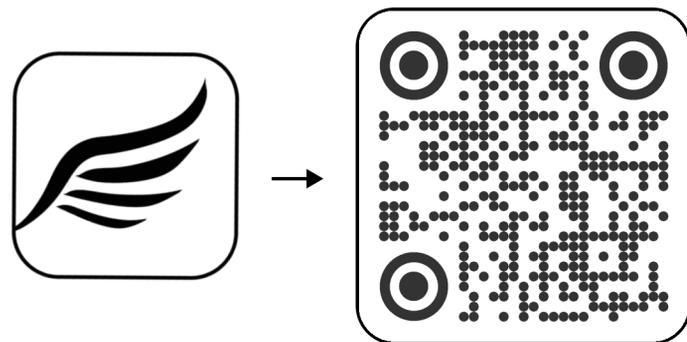


Download the Innov2Learn app from the App Store, the Play Store, or through our website: innov2learn.ca/app/

5



Open the Innov2Learn app.


6

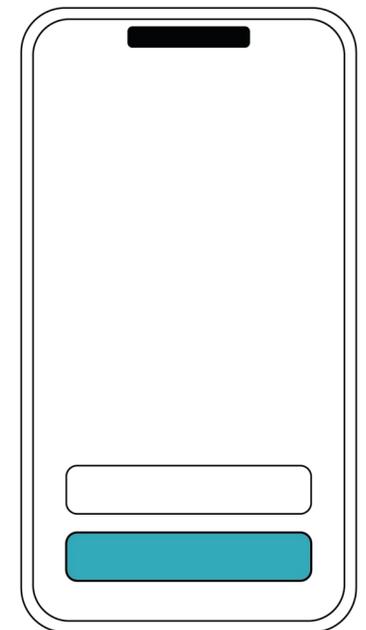

Enable Bluetooth and pair the device you'll be using with the SP Thermo Trigger.


SP Thermo Trigger - Instructions

7

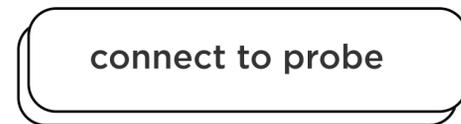


Once paired with the Thermo or SpotV II, click on "Connect to probe."

8

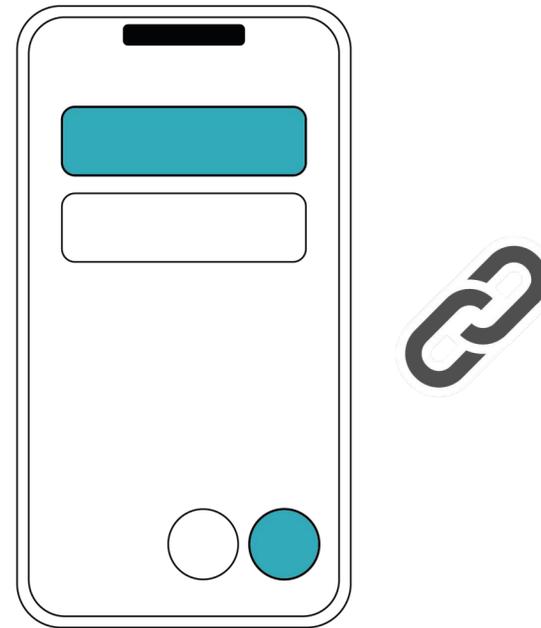


Pair and secure the SP Thermo Trigger connection.

9

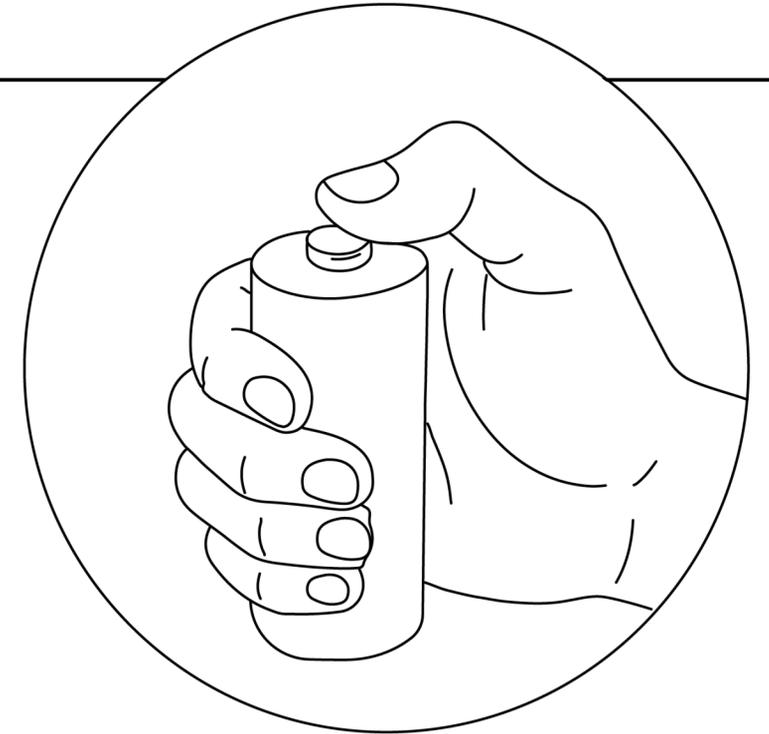

When ready, press the device button to enable detection in the app.

10

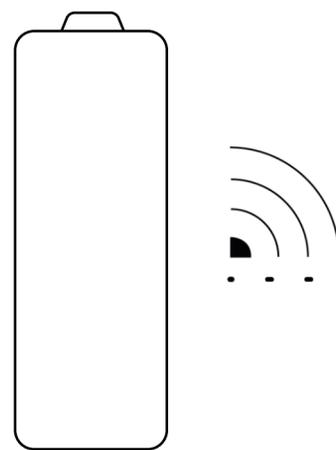

Start your simulation.

