



# SpotV II - Instructions

1

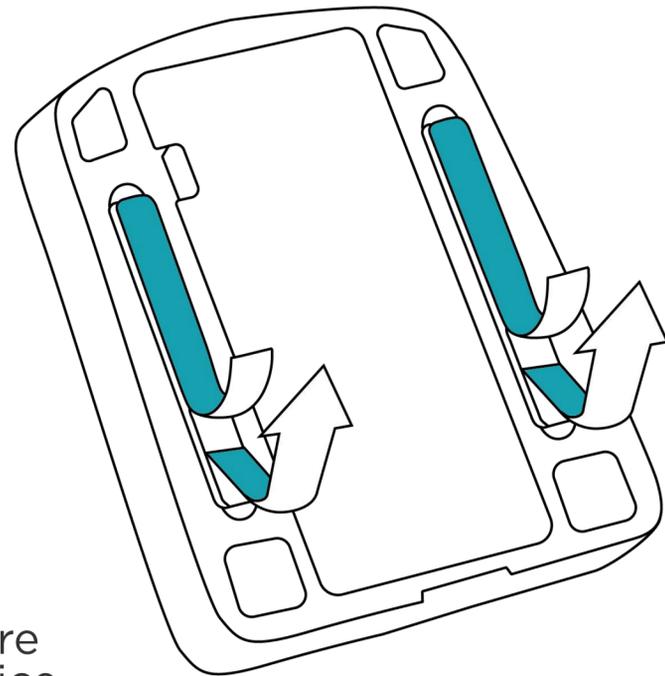


To secure the device, peel off the backing from the adhesive pads.

2

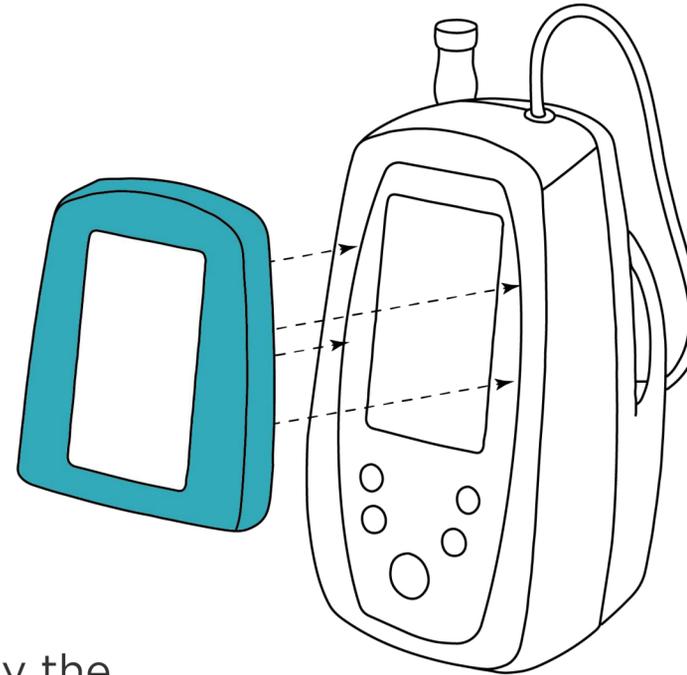


Apply the adhesive pads to the monitor to be simulated.

3

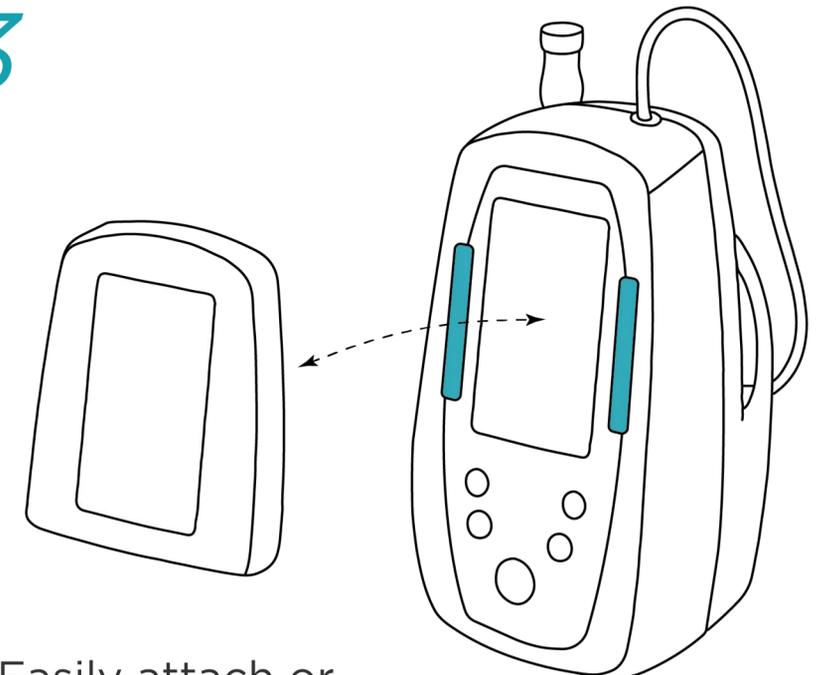


Easily attach or remove the device from its monitor.

4

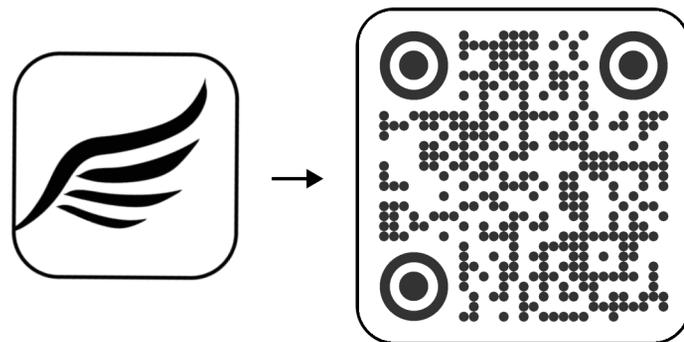


Download the Innov2Learn app from the App Store, the Play Store, or through our website: [innov2learn.ca/app/](http://innov2learn.ca/app/)

5

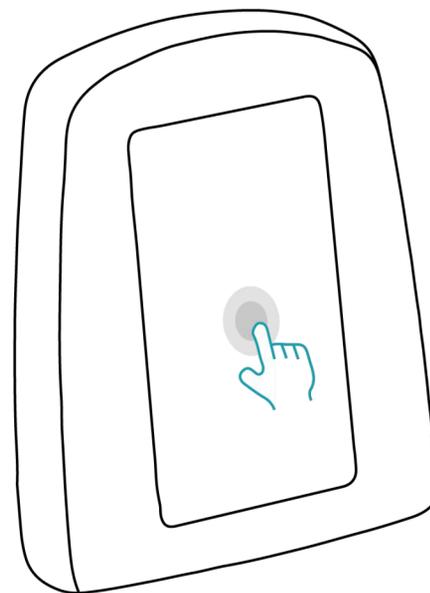

Double-tap the screen to turn on the device.

6

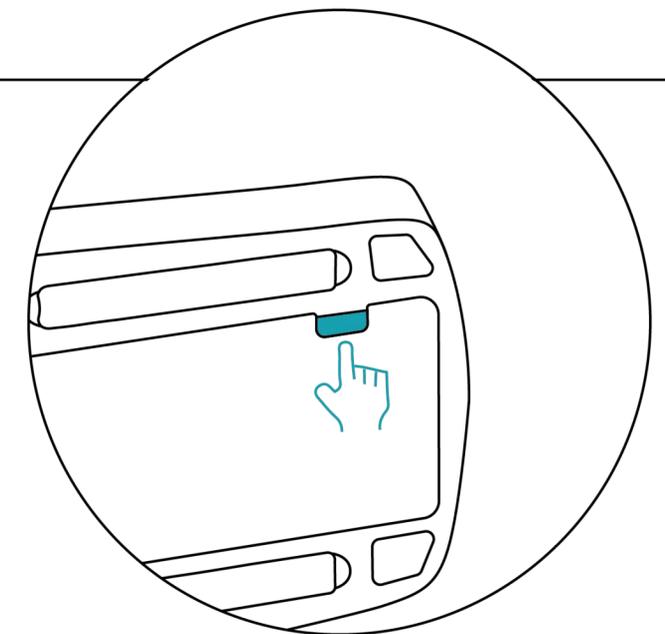

If the device is completely out of power, hold down the button on the back after recharging it.



# SpotV II - Instructions

7




Open the Innov2Learn app.

8

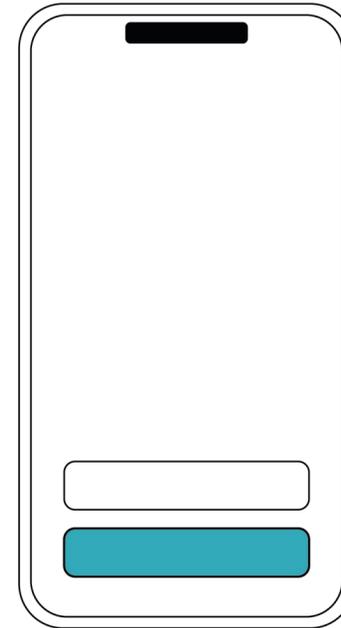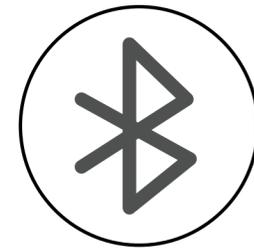


Turn on Bluetooth and click on "Connect".

9

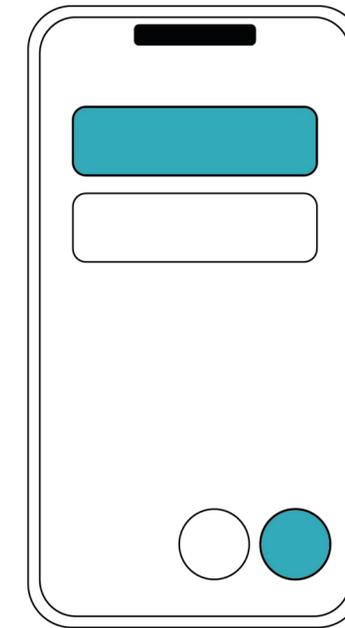

Select the device and press the pairing button.

10

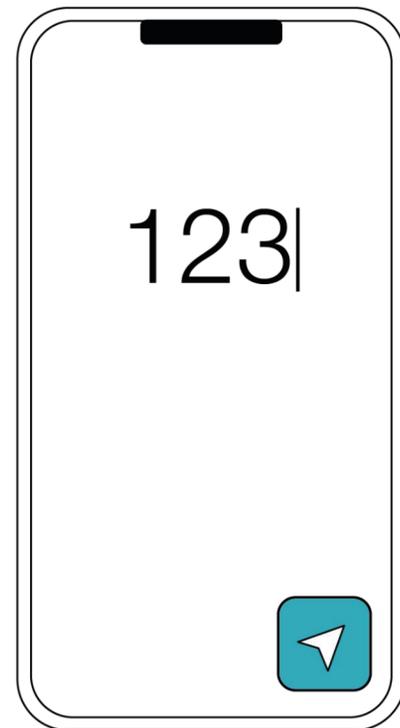

Start your simulation.

