



Master the finger-stick procedure with a needleless lancing device simulator producing lifelike blood drops.

Upon button press, a small drop of fake blood is pushed out of the device.

This needle-less kit is designed for use with both SPs and manikins. The included artificial blood is easily washable and non-staining, even on high-fidelity manikins.

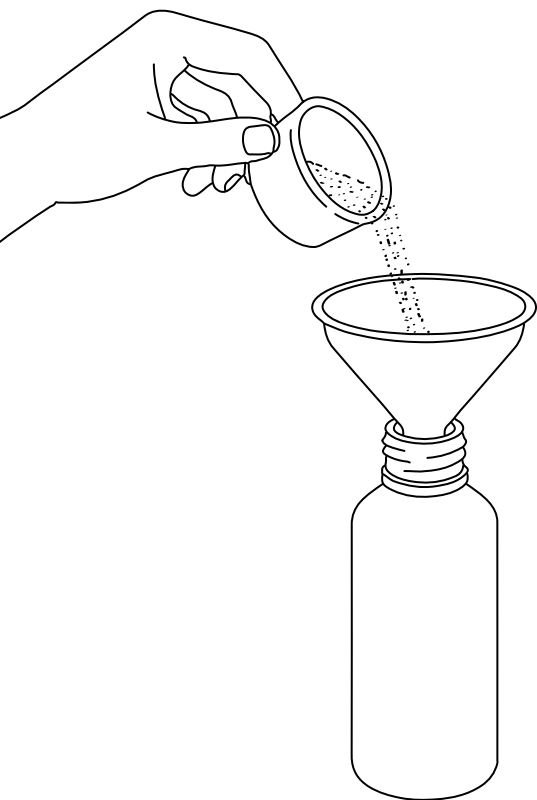

WHAT'S INCLUDED

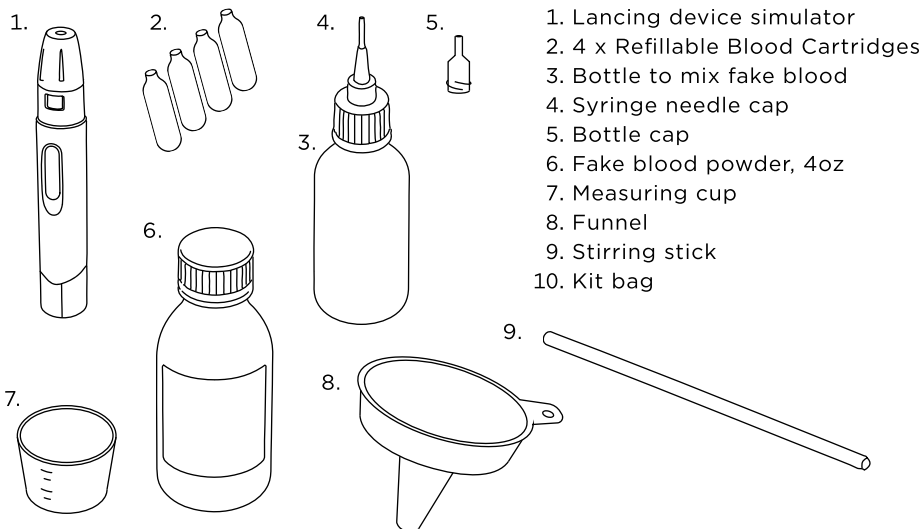

MIXING THE FAKE BLOOD

Easily prepare up to 2 Liters of realistic blood from the provided 4oz powder bottle.


1. Using the measuring cup, measure **10 ml** of fake blood powder. Place the funnel on top of the mixing bottle, then pour the measured powder into the bottle through the funnel.
2. Fill the bottle with warm water up to the shoulder line (the point where the bottle starts to narrow).
3. Stir the mixture thoroughly using the stirring stick until the powder is completely dissolved.
4. Close the bottle with the needle cap.

→ The fake blood is now ready to use.
Use within a few days and keep refrigerated.

READYING THE LANCING DEVICE SIMULATOR




The device comes with one cartridge already inserted. You can either refill it directly, or fill a spare cartridge and replace it using the stirring stick.

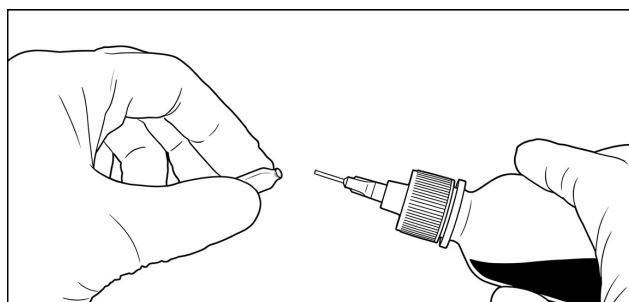

A - Filling and replacing a cartridge


1. Insert the needle of the fake blood bottle into a spare cartridge and fill it 3/4 full.
2. Open the top of the device and use the stirring stick to remove the empty cartridge.
3. Use the stirring stick to push in the filled cartridge. Ensure that the lip of the cartridge goes through the hole. Close the device

A -

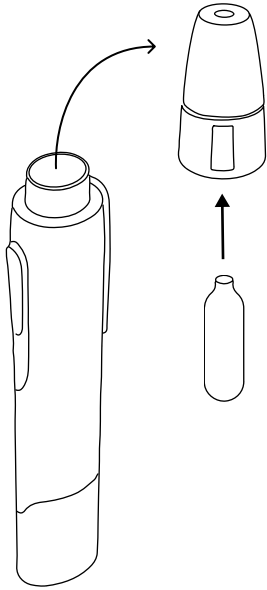

B - Filling the cartridge already in the device

1. Insert the needle of the fake blood bottle into the opening at the tip of the device.
2. Fill the cartridge until it is 3/4 full.

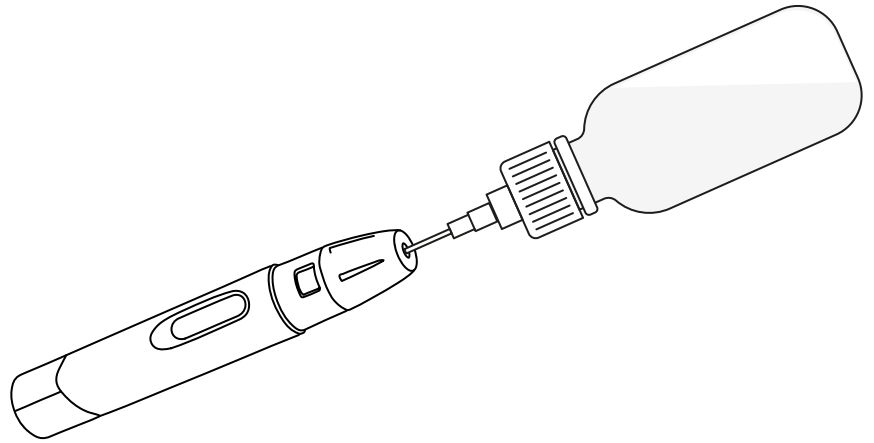

USING THE LANCING DEVICE

1. Ensure the fake blood cartridge in the lancing device is filled
2. Pull the rear cap backward until it clicks to arm the device.
3. Gently apply the head of lancing device on the skin surface.
4. Once ready, press the side button. A small drop of fake blood will appear.
5. To produce another drop, pull the rear cap back again to re-arm and repeat.

